



Connectivity

Patient monitoring, EHR/EMR/PDMS

To meet the ever-growing demand for data transferability in the hospital environment, Hamilton Medical ventilators offer a serial RS-232 interface and several protocols for connection to patient monitoring systems, Electronic Health Records (EHR), Electronic Medical Record systems (EMR), and Patient Data Management Systems (PDMS).

This document describes connectivity options for the most common EHR/EMR/PDMS systems.

Note that the information in this document may be changed or updated without notice. For further details, contact your Hamilton Medical sales representative.



Swiss
Quality

HAMILTON
MEDICAL
Intelligent Ventilation since 1983

Patient monitoring systems

HAMILTON-S1	HAMILTON-G5	HAMILTON-C1	HAMILTON-T1	HAMILTON-C2	HAMILTON-C3	HAMILTON-C6	Manufacturer	System	Protocol
✓	✓	✓	✓	✓	✓	✓	Dräger	Infinity	Polling
o	o	o	o	o	o	o	GE Healthcare	Datex-Ohmeda	
✓	✓	✓	✓	o	o	o	Masimo	Patient SafetyNet System and Iris Gateway System	Block
o	o	o	o	o	o	o	Maquet	Octanet	Polling
✓	✓	✓	✓	✓	✓	✓	Mindray	Beneview	Polling / Block
x	x	x	x	x	x	x	Nihon Kohden	BSM-4100K (v01-30 or later)	Polling
x	x	x	x	x	x	x	Nihon Kohden	BSM-5100K (v01-30 or later)	Polling
✓	✓	✓	✓	✓	✓	✓	Nihon Kohden	BSM-9101K (v12-06 or later)	Polling
✓	✓	✓	✓	✓	✓	✓	Nihon Kohden	BSM-6000K (v02-10 or later)	Polling
✓	✓	✓	✓	✓	✓	✓	Philips	IntelliVue (IntelliBridge)	VueLink Open
✓	✓	✓	✓	✓	✓	✓	Philips	IntelliVue (VueLink)	VueLink Open
o	o	o	o	o	o	o	Schiller	ARGUS	Polling
✓	✓	✓	✓	✓	✓	✓	Spacelabs Medical	Ultraview	Polling

✓ External device is able to read data from the Hamilton Medical device depending on the software version. The accuracy of the displayed monitoring data is not validated by Hamilton Medical.

x External device is not able to read data / no drivers have been developed.

o Contact the monitoring system manufacturer for the current status.

EHR/EMR/PDMS

HAMILTON-S1	HAMILTON-G5	HAMILTON-C1	HAMILTON-T1	HAMILTON-C2	HAMILTON-C3	HAMILTON-C6	Manufacturer	System	Protocol
✓	✓	✓	✓	✓	✓	✓	AGFA	ORBIS	Polling / Block
✓	✓	✓	✓	✓	✓	✓	Ascom	Digisat Suite	Polling
✓	✓	o	o	o	o	o	B-Simple	B-ICU.Care	Polling / Block
✓	✓	✓	✓	✓	✓	✓	Cerner	CareAware	Polling / Block
✓	✓	o	o	o	o	o	Chipsoft	CS-ECIS (Electronic Care Information System)	Polling
✓	✓	o	o	o	o	o	Clinicomp	Essentris	Block
✓	✓	✓	✓	✓	✓	✓	HIM	QCare	Polling
✓	✓	✓	✓	✓	✓	✓	Copra System	Copra	Polling
✓	✓	✓	✓	✓	✓	✓	Dräger	Integrated Care Manager (ICM)	Polling / Block
✓	✓	✓	✓	✓	✓	✓	GE Healthcare	Centricity™ Critical Care	Polling / Block
✓	✓	o	o	o	o	o	GE Healthcare	Unity Network ID	Polling
✓	✓	✓	✓	✓	✓	✓	iMDsoft	MetaVision	Polling / Block
o	o	o	o	o	o	o	IMESO	ICUData	Polling
✓	✓	✓	✓	✓	✓	✓	LOWTeq	LOWTeq-PDMS intensive care	Polling
✓	✓	o	o	o	o	o	McKesson	Horizon Clinicals® DAS	Polling
✓	✓	✓	✓	✓	✓	✓	NantHealth	DeviceConX™	Block
✓	✓	o	o	✓	o	o	Philips	ICCA (former ICIP Release D.03 ¹⁾)	Polling
✓	✓	o	o	o	o	o	PICIS	Picis Critical Care Manager	Polling
✓	✓	✓	✓	✓	✓	✓	Precept Health Ltd.	ICU Care	Polling / Block

✓ External device is able to read data from the Hamilton Medical device depending on the software version. The accuracy of the displayed monitoring data is not validated by Hamilton Medical.

o Contact the EHR/EMS/PDMS monitoring system manufacturer for the current status.

1) With service pack 1 or higher.

Connectivity solutions middleware

HAMILTON-S1	HAMILTON-G5	HAMILTON-C1	HAMILTON-T1	HAMILTON-C2	HAMILTON-C3	HAMILTON-C6	Manufacturer	System	Protocol
✓	✓	✓	✓	✓	✓	✓	Bridge-Tech Medical	Device Connectivity Solution (DCS)	Polling / Block
✓	✓	✓	✓	✓	✓	✓	Capsule Technologie	SmartLinx Device Driver Interface Library (DataCaptor)	Polling / Block
✓	✓	✓	✓	✓	✓	✓	Cardiopulmonary Corp.	Bernoulli® Enterprise	Block
o	o	o	o	o	o	o	CareTrends	careTrends® software platform	
o	o	o	o	o	o	o	Extension Healthcare	Extension Engage™	
✓	✓	✓	✓	✓	✓	✓	MedKom	MCare	Polling / Block
✓	✓	o	o	o	o	o	NANTHEALTH iSirona	DeviceConX	Polling
o	o	o	o	o	o	o	Nuvon	Nuvon Gateway	
✓	✓	✓	✓	✓	✓	✓	Philips	IntelliBridge	VueLink Open

✓ External device is able to read data from the Hamilton Medical device depending on the software version. The accuracy of the displayed monitoring data is not validated by Hamilton Medical.

o Contact the monitoring system manufacturer for the current status.

Connectivity between EHR/EMR/PDMS and middleware

EHR/EMR/PDMS system	Middleware					
	Bridge-Tech Medical	Capsule Technologie	Cardiopulmonary Corp.	CareTrends	NANTHEALTH iSirona	Nuvon
Allscripts	o	o	o	o	✓	o
CPSI	✓✓	o	o	o	o	✓
Ecare NEBO	o	o	o	o	o	o
Eclypsis (Allscripts)	✓✓	o	o	✓	o	✓
Epic	✓✓	✓✓	o	✓	✓	✓
HMS Ambulatory Care	✓✓	o	o	o	✓	o
McKesson	✓✓	✓✓	o	✓	o	✓
MEDITECH	✓✓	✓✓	o	✓	✓	✓
Medsphere Open Vista	o	o	o	o	o	o
Orion	o	o	o	o	o	o
Siemens Sorion	✓✓	✓✓	o	o	✓	✓

✓✓ Connectivity between EHR/EMR and middleware, as well as between middleware and the Hamilton Medical device. See the table "Connectivity solutions middleware" on the previous page.

✓ EHR/EMR should be able to communicate with middleware. The accuracy of this information is not validated by Hamilton Medical.

o Contact the middleware manufacturer for the current status.

Other systems

HAMILTON-S1	HAMILTON-G5	HAMILTON-C1	HAMILTON-T1	HAMILTON-C2	HAMILTON-C3	HAMILTON-C6	Manufacturer	System	Protocol
✓	✓	✓	✓	✓	✓	✓	Mallinckrodt Pharmaceuticals	INOmax® DSIR®	Polling
✓	✓	✓	✓	o	✓	✓	EKU	NO-A, EZ-KINOX, CARDINO	Block

✓ External device is able to read data from the Hamilton Medical device depending on software version. The accuracy of displayed monitoring data is not validated by Hamilton Medical.

o Contact the monitoring system manufacturer for the current status.

Distributed alarm systems

HAMILTON-S1	HAMILTON-G5	HAMILTON-C1	HAMILTON-T1	HAMILTON-C2	HAMILTON-C3	HAMILTON-C6	Manufacturer	System	Protocol
✓	✓	x	x	x	x	✓	ASCOM	Digistat Smart Central Plus	Block

✓ External device is able to read data from the Hamilton Medical device depending on software version. The accuracy of displayed monitoring data is not validated by Hamilton Medical.

o Contact the monitoring system manufacturer for the current status.

x Not available

Manufacturer:

Hamilton Medical AG

Via Crusch 8, 7402 Bonaduz, Switzerland

☎ +41 (0)58 610 10 20

info@hamilton-medical.com

www.hamilton-medical.com

ELO20150228N.04

Specifications are subject to change without notice. Some features are options. Not all features/products are available in all markets. For all proprietary trademarks (®) and third-party trademarks (®) used by Hamilton Medical AG see www.hamilton-medical.com/trademarks.
© 2021 Hamilton Medical AG. All rights reserved.