



As individual as they are.

GoLife for Men and **GoLife for Women** are as unique in shape and structure as the men and women who wear them.

PHILIPS

RESPIRONICS

sense **and** simplicity

GoLife gives men and women the unique fit each one deserves.

GoLife for Men and GoLife for Women are our first nasal pillows masks to be built around the unique features of men and women. The result is a gender-specific fit that provides tremendous stability.

At first glance it's clear these two masks share a common platform, yet they're quite different. The most obvious distinctions are the attractive colors that differentiate GoLife for Men and GoLife for Women.

But the real beauty is more than skin deep.

A closer look reveals the true design advantages of each. GoLife for Men is designed specifically to fit the larger contours of the male facial structure, while GoLife for Women is developed with a distinctly smaller frame and headgear to more closely fit feminine facial dimensions.

The GoLife platform is built around a simple concept with few parts. The unique soft facial contour arms gently hug the face to maintain stability and a secure seal even as the patient moves. Self-adjusting, optimally angled nasal pillows allow a one-step fit. Comfortable, straightforward headgear comes in two sizes – one for men, the other for women.

Compare GoLife for Men and GoLife for Women and soon you'll discover the unique fit of GoLife masks.

Simple fitting steps

Pull the headgear over your head.
Place the cushion into your nostrils.
Select a cushion that fills your nostrils comfortably without any gaps between the cushion and the walls of your nostrils.



Adjust the side straps.
Do not overtighten.



If needed, adjust the top headgear strap. Allow the headgear to rest just above the ears like a pair of eyeglasses. The back strap will sit at the base of the head.



Connect the tubing to the device.
Turn on the airflow. Make final adjustments while laying down.



To attach a cushion, squeeze the sides so it flexes inward. Place the cushion over the tubing so that it rests on the lower lip of the mask. Push the round nub on one end of the cushion into the circle on the frame and repeat on the other side.




GoLife mask frames and headgear are designed differently to fit men and women.

Soft facial contour arms embrace the face with stability by hugging the cheekbones.



Self-adjusting, optimally angled nasal pillows eliminate the need for manual adjustment and provide a seal that's flexible.

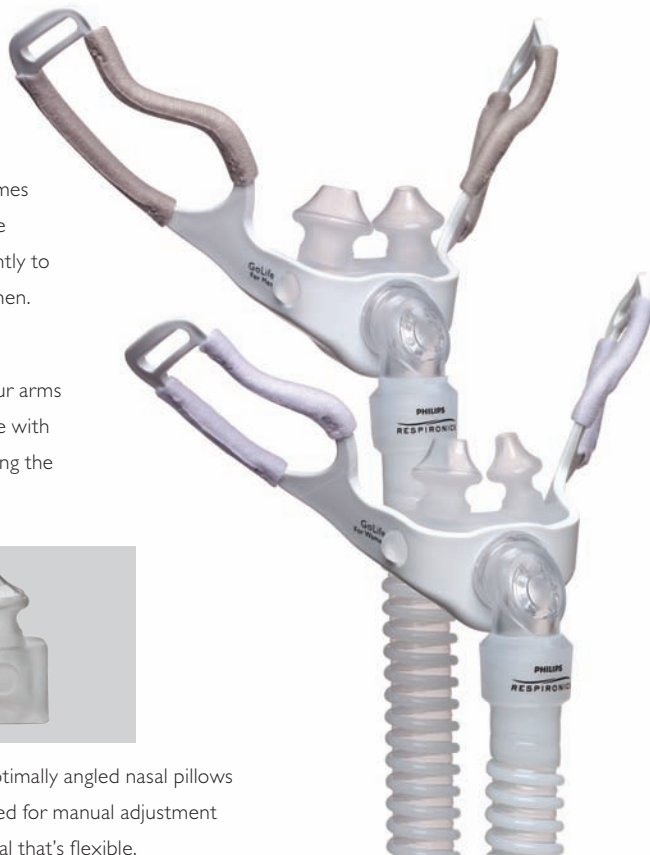

Diffused airflow for quiet exhalation.

Long, multi-directional 45.7 cm tube with 360 degree swivel maintains seal during patient movement.



GoLife for Women has smaller headgear that sits lower on the back of the head to prevent slippage and to help provide a better fit.

Ordering information	
GoLife for Women mask	Part number
FitPack: mask with headgear plus petite, small, and medium cushions	1082580
GoLife for Women headgear	1082581
GoLife for Men mask	Part number
FitPack: mask with headgear plus small, medium, and large cushions	1073331
GoLife for Men headgear	1073336
Replacement parts and accessories	Part number
Replacement cushion, petite	1073332
Replacement cushion, small	1073333
Replacement cushion, medium	1073334
Replacement cushion, large	1073335
Tube management clip, 2 per pack	1073337
Swivel tubing (18 inch/45.7 cm) with exhalation port	1078407

**Philips Healthcare is part of
Royal Philips Electronics**

How to reach us

www.philips.com/healthcare
healthcare@philips.com

Asia

+49 7031 463 2254

Europe, Middle East, Africa

+49 7031 463 2254

Latin America

+55 11 2125 0744

North America

+1 425 487 7000

800 285 5585 (toll free, US only)

Philips Respiroics
1010 Murry Ridge Lane
Murrysville, PA 15668

Customer Service
+1 724 387 4000
800 345 6443 (toll free, US only)

Philips Respiroics International
Headquarters
+33 1 47 28 30 82

Philips Respiroics Asia Pacific
+65 6882 5282

Philips Respiroics Australia
+61 (2) 9666 4444

Philips Respiroics China
+86 400 820 6665
+86 800 820 6665

Philips Respiroics Deutschland
+49 8152 93 06 0

Philips Respiroics France
+33 2 51 89 36 00

Philips Respiroics Italy
+39 039 203 1

Philips Respiroics Sweden
+46 8 120 45 900

Philips Respiroics Switzerland
+41 6 27 45 17 50

Philips Respiroics United Kingdom
+44 800 1300 845

www.philips.com/respiroics

Respiroics, FitPack, and GoLife are trademarks of Respiroics, Inc. and its affiliates. All rights reserved.

Please visit www.philips.com/golife



© 2011 Koninklijke Philips Electronics N.V.
All rights are reserved.

Philips Healthcare reserves the right to make changes in specifications and/or to discontinue any product at any time without notice or obligation and will not be liable for any consequences resulting from the use of this publication.

CAUTION: US federal law restricts these devices to sale by or on the order of a physician.
Broudy NA 08/22/11 12NC 45296275831 MCI 4104331 PN 1092676